

AED Plus Trainer2

Practice an AED Plus Rescue with
Total Instructor Control



ITG Instructor Training Ltd
info@itg-first-aid.com
Authorised Dealer

An accurate, well-controlled, and simple simulation of what it's like to use an AED Plus® in a real rescue.

ZOLL
Advancing Resuscitation. Today.®

AED Plus Trainer2 includes:

- AED Plus Trainer2 training AED
- Wireless remote controller
- One set of AED Plus Training **CPR-D•padz**[®]
- One set of AED Plus Trainer **CPR-D•padz** reusable gels
- 4 D cell batteries
- 2 AA cell batteries
- Operator's Guide

ZOLL Medical Corporation Worldwide Headquarters

269 Mill Road
Chelmsford, MA 01824
978-421-9655
800-348-9011

ZOLL International Offices

Australia
+61 2 8424 8700
www.zoll.com.au

Austria
+43 650 413 6222
www.zollmedical.at

Canada
905-629-5005
www.zoll.com

China
+86 21 6226 9505
www.zoll.com

France
+33 (1) 30 05 14 98
www.zoll.fr

Germany
+49 2236 87 87 0
www.zollmedical.de

Middle East and Africa,
+96 278 800 4411
www.zoll.com

The Netherlands
EU Authorized Representative
+31 (0) 488 41 11 83
www.zoll.nl

Russia
+7495 936 2338
www.zoll.com

United Kingdom
+44 (0) 1928 595 160
www.zoll.com

For subsidiary addresses and fax numbers, as well as other global locations, please go to
www.zoll.com/contacts.

Complete Simulation Control at the Instructor's Fingertips

The ZOLL[®] AED Plus Trainer2 allows CPR and first aid instructors to have students experience what it's like to save a life with the AED Plus. Choose from four different fixed rescue scenarios or have complete manual control of the entire rescue. A wireless remote controller enables the control of multiple AED Plus Trainer2s.

With this simple remote controller, instructors can set up a rescue scenario and download to the AED Plus Trainer2 in seconds. The student pushes the on-off button on the Trainer2 unit to begin. The instructor then determines whether to shock on the next simulated heart analysis or not, whether the pads have been properly attached or not, and when the Trainer2 unit should say "PUSH HARDER" and "GOOD COMPRESSIONS."

A simple pause button lets the instructor stop the rescue altogether, confer with the student, and then pick up exactly where the simulated rescue left off.

Protocols conform to International Liaison Committee on Resuscitation (ILCOR) standards including American Heart Association (AHA) and European Resuscitation Council (ERC) 2005 Guidelines.

Trainer 2 easily switches from AHA to ERC 2005 Guidelines



AED Plus Trainer2 Specifications

Dimensions: 5.25" H x 9.5" W x 11.5" D (13.3cm H x 24.1cm W x 29.2cm D)

Weight: 4 lbs 14 oz (2.2 kg)

Power: 4 D cells (included) for Training AED (Good for more than 100 hours of training)

2 AA cells (included) for remote controller

AC adapter not available—battery operated only.



ITG Instructor Training Ltd
info@itg-first-aid.com
Authorised Dealer



Specifications subject to change without notice.

©2007 ZOLL Medical Corporation. All rights reserved. "Advancing Resuscitation. Today.", AED Plus, CPR-D•padz, and ZOLL are registered trademarks of ZOLL Medical Corporation.

Printed in U.S.A. 120710 9656-0202

ZOLL[®]
Advancing Resuscitation. Today.[®]